DENTAL MAKERS

LIGHTBUILDER^{4K}
2.0

User Manual

LIBER
2.0

AVVERTENZE

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www.dentalmakers.it

WARNINGS

This 3D Printer is intended for Professional Use and is therefore recommended to be used only by Qualified Personnel.

Before using this 3D Printer it is necessary to read the instructions and become familiar with the necessary PPE to work safely with resins.

In the processing phases this 3D printer uses UV light to polymerize the resin.

Avoid staring directly into UV light, it could be dangerous for your eyes.

It is advisable to wear protective glasses with UV filter lenses; those with EN 172 approval are suitable for use.

This printer uses a stepper motor and a worm screw to move the print bed.

Its movement is very slow but it is still advisable not to bring your hands near the moving parts during the processing phase.

During the entire printing process, always keep the front door closed to avoid contact with mechanical parts and avoid exposure to UV rays, as well as contact with uncured resin.

The resin in liquid form, therefore before the polymerization process, is harmful to the human body because it contains volatile substances harmful to the respiratory tract.

Use

nitrile gloves

protective glasses (compliant with EN 172) mask suitable for gas filtering

It is recommended to install and use the 3D Printer in a well-ventilated place or under a fume hood.

WARNINGS

Clean all components of this 3D Printer with particular care, as the resin could, in the long term, affect the integrity of some parts of the machine, compromising its normal use.

In the event of a resin leak inside the 3D Printer, remove power and contact Technical Support in DENTAL MAKERS S.r.l.

This machine has a heating system with maximum temperature of +30°C.

In case of malfunction, remove the power supply contact Technical Assistance at DENTAL MAKERS S.r.l.

In the event of tampering, unauthorized modification or removal of components of the 3D Printer, the Manufacturer is not considered responsible for any damage to people and/or things.

Otherwise only follow the Manufacturer's instructions will be considered an improper use of the 3D Printer e all warranty repair clauses will be void.

Keep the 3D Printer away from heat sources as some types of resins used are easily flammable.

In case of prolonged non-use of the 3D printer we recommend remove the power cord and proceed to thorough cleaning and removal of resin residues.

If parts are damaged and need to be replaced, contact Assistance technique in DENTAL MAKERS S.r.l. to receive only original spare parts.

PACKAGE CONTENTS

LightBuilder 4K 2.0

Resin Tank

Print Platform

Wire Cutters

Hex Keys

Empty USB Disk
File Transfer

USB Disk
"Utilities Drive"


ACDC Adapters 24V 10A

Yellow spatula for mixing resin

Metal spatula for removing
the models from the print platform

3D PRINTER

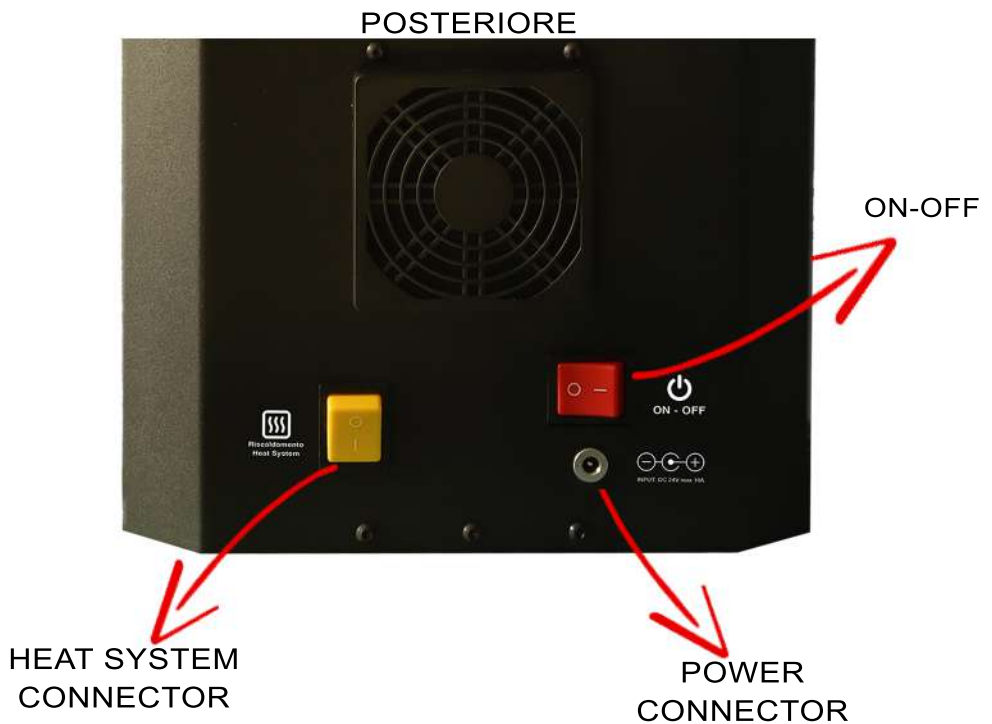3D PRINTER

Equipment size	300x300x500 mm
Building size	143x89x150 mm
LCD resolution	4098x2560 (4k mono)
User Interface	5 inch touch screen
Layer	0.025 - 0.1 mm
Molding speed	50mm/h
Molding band	405nM
Slicer Software	Chitubox
File transimission	Wifi - USB
Data Format	Stl, Obj, Slc
Voltage	240V/110V 50Hz
Power	200W
Weight	17 Kg

INSTALLATIONS

Make sure that the contents of the package are intact and complete, otherwise contact customer support.

After receiving the Lightbuilder 4K 2.0 the user must just remove the packaging and connect the power to the connector located on the back of the 3D printer.

The Chitubox printing software installation will come subsequently ALWAYS carried out by Dental Makers via remote connection software or by your local distributor

Remote connection software

<https://anydesk.com/it/downloads>

INSTALLATIONS

During the installation of the software you will be provided with all the guidelines for the correct functioning of the machine and the printing software.

It will be possible to find all this information in the form of video tutorials in the "Utilities Drive" USB stick supplied with the 3D printer.

On the USB stick you will find two distinct folders: Machine Tutorial and Software Tutorial, respectively with video guide for the operations to be performed on the printer and in chitubox software. The machine's firmware is also present, the parameters and the printing software, elements that could facilitate our possible remote assistance.

We therefore recommend storing the "Utility Drive" carefully. In case of loss, we will provide a new USB stick at your expense contact.

By scanning the QR you will still be able to download all the material listed above which may be updated over time

SPARE PARTS AND ACCESSORIES

All components of the Lightnuilder 4K 3D printer are described in the component list received with the machine.

We specify that in the event of failure of any component, Dental Makers or the local distributor will have to collect the machinery for repair, except for the transformer which can be shipped to the customer.

There are two accessories that can be purchased and installed directly from the customer, to be understood as additional components and not as spare parts:

Print Platform

Resin Tanks

For the correct use of consumables (Resins), read carefully the information on the package or leaflet illustrative both in the case of Dental Makers materials and other brands

DENTAL MAKERS

PRODUCT COMPLIANCE

with the essential requirements of European directives

Simplified EU declaration of conformity
article 10, paragraph 9 and in accordance with Annex VII of Directive 2014/53/EU

The manufacturer Dental Makers declares that the type of radio equipment
Dental Makers Lightbuilder 4K complies with Directive 2014/53/EU

The nameplate on the 3D printer is like the following image

Scan the QR code to view the statement
extended compliance
(draft_EU-DoC_REd includes Turkey assessment)

CONTACTS

Dental Makers
+39 0775 163 0167
info@dentalmakers.it

Administration-Commercial
+39 380 389 5852

Technical assistance
+39 389 945 3449

Mobile numbers are also available
for WhatsApp contact

www.dentalmakers.it

Produced by:

DENTAL MAKERS

Dental Makers Via Arelle 9, 03020 Vallecorsa (FR) Italia

tel. +39 0775 163 0167 web. www.dentalmakers.it mail. info@dentalmakers.it